Schulterchirurgie Shoulder Surgery

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CE 0482

Waldemar Link GmbH & Co. KG

Barkhausenweg 10 · 22339 Hamburg

Postfach 63 05 52 · 22315 Hamburg

Telefon: +49 40 53995-0 · Fax: +49 40 5386929

E-Mail: info@linkhh.de · Internet: www.linkhh.de

LINK[®] Schulterchirurgie

LINK[®] Shoulder Surgery

02	Beschreibung / Indikationen	Description / Indications
03	Instrumentarium	Instrument Set
05	Instrumente	Instruments
14	Zusätzliche Instrumente	Additional Instruments
17	Zubehör	Accessories
18	OP-Technik	Surgical Technique
25	Index	Index
	Wichtige Hinweise	Important Information

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The goal of operative treatment of posttraumatic recurrent glenohumeral luxation is to restore stability by repairing the traumatic lesion, returning structures to their anatomic locations and restoring the physiological range of motion.

The surgical procedure by which this can be achieved is the

- **Bankart Procedure**

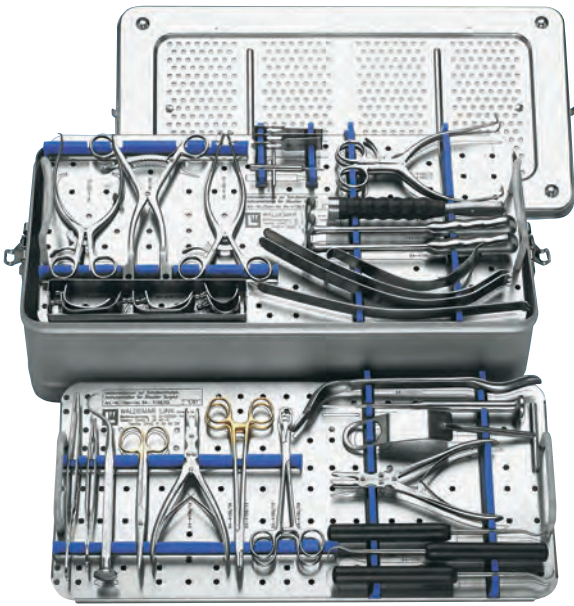
In this procedure, it is necessary to expose the anatomic structures individually and to suture them in the original location after reattachment of the capsule to the anterior rim of the glenoid cavity. This is the only way to obtain equilibrium of the muscle actions with full joint mobility.

A specially designed instrument set is required to perform a quick and tissue-sparing procedure. The instrument set featured in this Surgical Technique meets these requirements and can also be used for many other forms of shoulder surgery.

Other **indications** for the Shoulder Surgery instrument set:

- Capsuloplasties for multidirectional instabilities
- Subacromial decompression (Neer procedure)
- Calcium excision in the case of Calcific Tendinitis
- Shoulder replacement, fracture repair

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Percussion Awls

with square point and striking head, curved, 220 mm

Art.-Nr. Item No.	Größe	Size
64-4160/20	klein	small
64-4160/05*	mittel	medium
64-4160/22	groß	large
64-4160/23	extragroß	x-large

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* Size "medium" included in Instrument Set

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For transosseous humeral sutures. Use in conjunction with trocar needles page 20.

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Penetrating Awl with square point, curved, 235 mm

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Crochet Hook Suture Passer, modified version:

- shaft area 20 mm reduced
- handle 30 mm reduced
- total length: 185 mm

64-4160/08

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Hollow Lumen Suture Passer, curved, 235 mm

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Hollow Lumen Suture Passer, modified version:

- shaft area 20 mm reduced
- handle 30 mm reduced
- total length: 185 mm

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Penetrating Gouge, 3 mm wide, 220 mm.
For perforating the glenoid rim or neck of the scapula.

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Humerus head retractor, forked, inward-
angled blunt tips, strongly curved, 180 mm

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Fukuda-Kujat's Humeral Head
Retractor, 35 mm, wide blade, 225 mm

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Richardson Retractor,
20 mm wide x 55 mm deep, 235 mm

For retracting the tissue when opening
the deltopectoral interval.

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Kölbel Glenoid Retractor,
double-pronged, angled tips, 300 mm

Art.-Nr. Item No.	Breite Width	Version
64-4160/13	18 mm	gerade straight
64-4160/32	15 mm	stark gebogen strongly curved
64-4160/33	23 mm	stark gebogen strongly curved

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Kölbl Soft Tissue Retractor, single-pronged, straight, very fine tips, 190 mm

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Kölbl Soft Tissue Retractor, single-pronged, angled, 190 mm

For retraction when using the deltopectoral approach.

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Kölbl Soft Tissue Retractor, double-pronged, angled, 190 mm

For retraction when using the deltopectoral approach.

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Osteotome Set

with three exchangeable blades
(10, 16 and 25 mm) and one key,
200 mm

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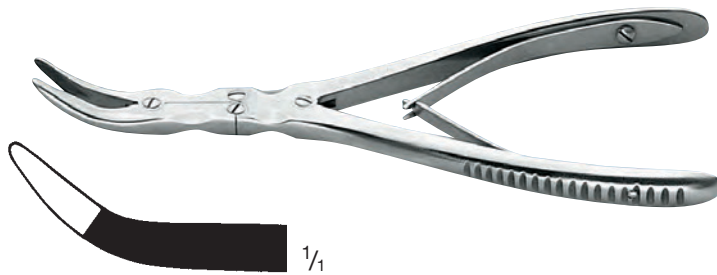
Bone Nibbling Rongeur,
double-action, slightly curved, 5 mm
jaw width, 190 mm

For roughening the rim of the glenoid
at the side of the Bankart lesion and
for preparing the humerus for rotator
reconstruction. Also for modeling the
acromion.

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Köbel Tenectomy Rongeur, double-action, strongly curved, 2.5 x 5.0 mm jaw width, 180 mm

For removing tendon residue and for deepening the neck of the humerus in tendon refixation.

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Skid Lever, (bone lever and skid curette), double-ended, 280 mm

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Köbel Self-Retaining Glenoid Retractor, with ball snap-fit, slightly curved, with 3 pairs of blades, 200 mm

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Capsule and Tendon Scissors
curved, with serrated carbide inserts,
170 mm

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Kölbel Tissue Forceps
1 x 2 teeth and serrated jaws, heavy
pattern can securely grasp capsular
membrane, 210 mm

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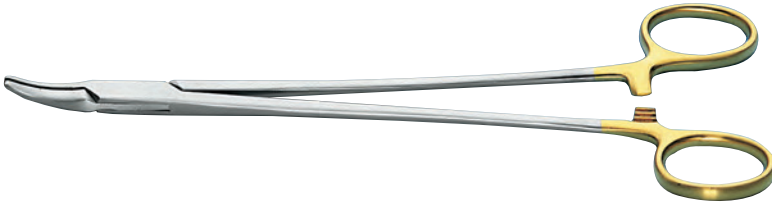
Bonney Thumb Forceps,
1 x 2 teeth and serrations,
175 mm

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Heaney Needle Holder
curved, carbide inserts, 205 mm

For transosseous suturing of the cap-
sule and for passing the strong Mayo
needle through the bone canals of the
glenoid rim

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New Orleans Needle Holder
carbide inserts, 180 mm

For tendon and transosseous suturing
during reconstruction of the rotator
cuff.

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Kölbel Subscapular Spreader,
double-pronged, angled arms, 190 mm

For retraction when using the delto-pectoral approach.

Art.-Nr. Item No.	Maße Dimensions	Ausführung Version
64-4160/18	20 x 12 mm	klein small
64-4160/19	25 x 15 mm	groß large

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Dahmen-Lexer Gouge
25 mm wide, 220 mm

For partial resection of the acromio-clavicular joint.

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Mallet
with linen phenolic handle,
Ø 30 mm, 480g, 240 mm

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
Humerus Head Retractors,
american model

Art.-Nr. Item No.	Breite (Zoll) Width (inch)	Länge Length
64-4163/01	3 / 8"	255
64-4163/02	1 / 2"	255
64-4163/03	3 / 4"	255
64-4163/04	1"	355

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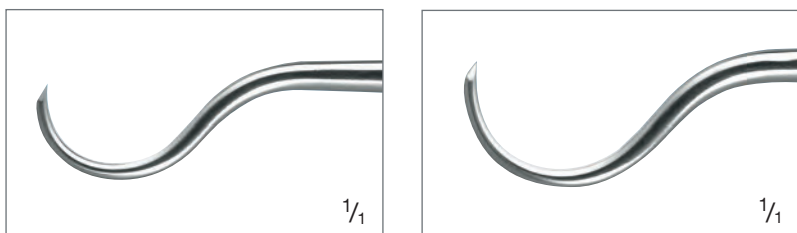Penetrating Awls,
american model, curved, 235 mm


Art.-Nr. Item No.	Radius
64-4161/61	9,5 mm
64-4161/62	11,5 mm

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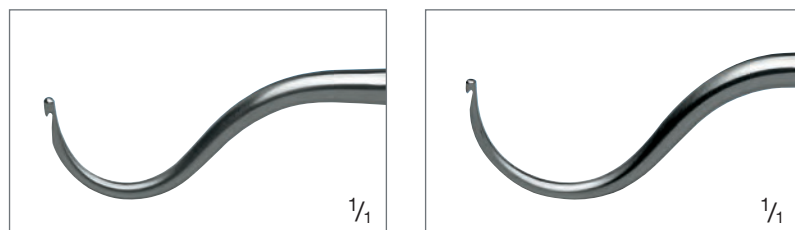Crochet Hook Suture Passer,
american model, curved, 235 mm

Art.-Nr. Item No.	Radius
64-4161/71	9,5 mm
64-4161/72	11,5 mm

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The awls and suture passers 64-4161/61, 62 and 64-4161/71, /72 differ from the instruments 64-4160/06 and /07 in the size of their tips. The curvature of these instruments corresponds to the curvature of the percussion awls 64-4160/20 and /22.

Nicht im Instrumentarium 64-4158/01 enthalten - not included in Instrument Set 64-4158/01

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Köbel Self Retaining Glenoid Retractor, with ball snap-fit, slightly curved, with 3 pairs of blades, 200 mm

Bestehend aus:
Consisting of:

Art.-Nr. Item No.	Ausführung Version	Tiefe x Breite Depth x Width
64-4161/02	Spreizer ohne Valven Retractor without blades	200 mm
64-4161/07	Valven kurz Blades, short	36 x 18 mm
64-4161/08	Valven lang Blades, long	53 x 18 mm
64-4161/09	Valven extralang Blades, extra long	68 x 18 mm

Einzelkomponenten zum Meißel-Set
Individual components for Osteotome Set
(64-4159/16)

64-4159/93 Griff, einzeln inkl. Schraube
Handle, single including Screw

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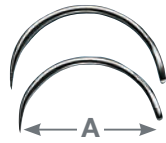
The Needles are ideal for transosseous sutures, i.e. for passing No. 2 and No. 3 sutures through the rim of the glenoid.

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Trocar Point Needles
with standard eye, pack of 24

Trokarnadel Trocar Point Needle		Ganzmetallpfriem Percussion Awl	
	64-4160/50	entspricht corresponds to	64-4160/20 klein/small 
	64-4160/51	entspricht corresponds to	64-4160/05 mittel/medium 
	64-4160/52	entspricht corresponds to	64-4160/22 groß/large 
	64-4160/53	entspricht corresponds to	64-4160/23 extragroß/x-large 

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
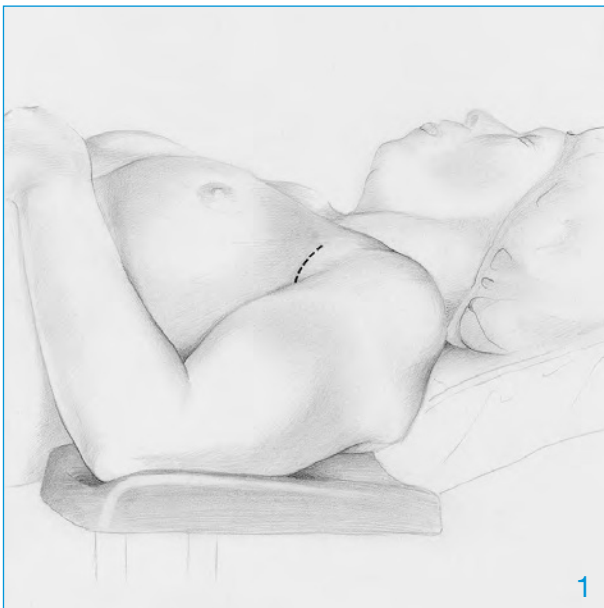
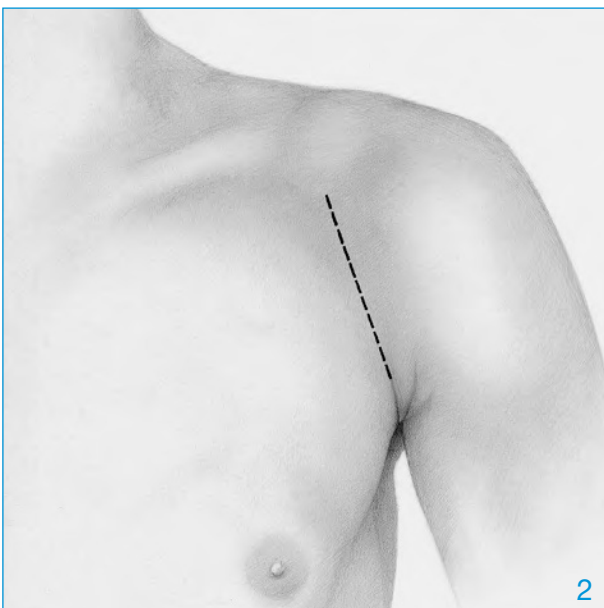
The upper arm is positioned on an arm rest that extends no further than the elbow. If the table is wide, the upper arm is positioned similarly on a folded drape leaving the shoulder exposed. This allows the surgeon to hold the arm if so desired and the humeral head to be shifted to dorsal. No cushion is placed under the scapula; by placing a drape, folded repeatedly to a thickness of 1 – 1.5 cm, medial of the medial border, the scapula can be pushed outwards, which can sometimes be advantageous.

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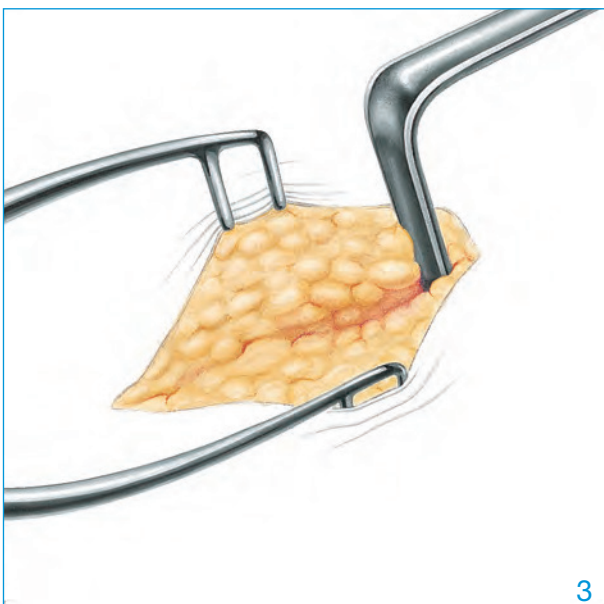
The skin incision is made from the coracoid to one of folds above the anterior edge of the axilla. The skin incision does not have to be as long for most women.

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
Retraction begins with the double-pronged Kölbel Soft Tissue Retractor and a Richardson Retractor.

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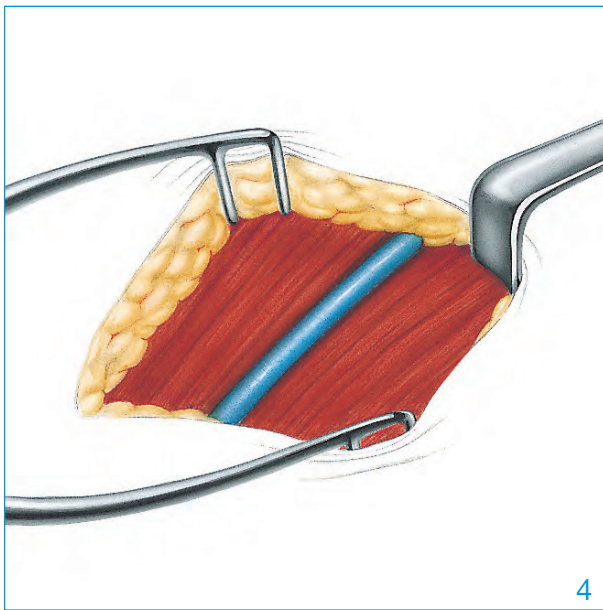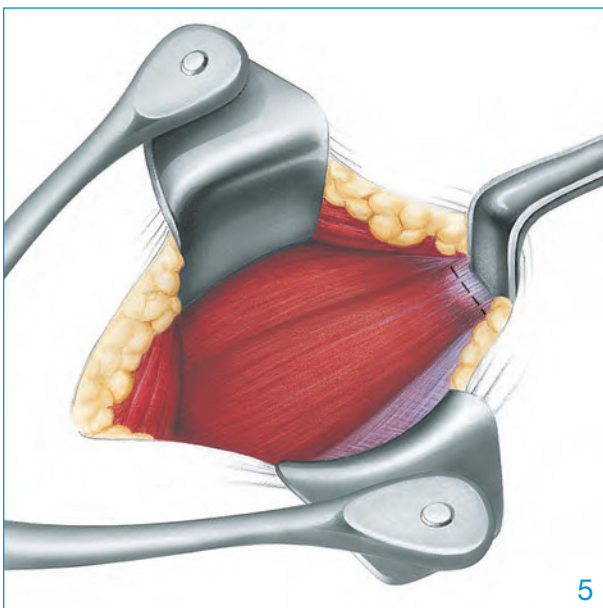
The cephalic vein runs obliquely to the skin incision from top right to bottom left. Access is via the deltoideopectoral sulcus with the cephalic vein to lateral. This exposes the coracoid and the lateral border of the coracobrachial muscle.

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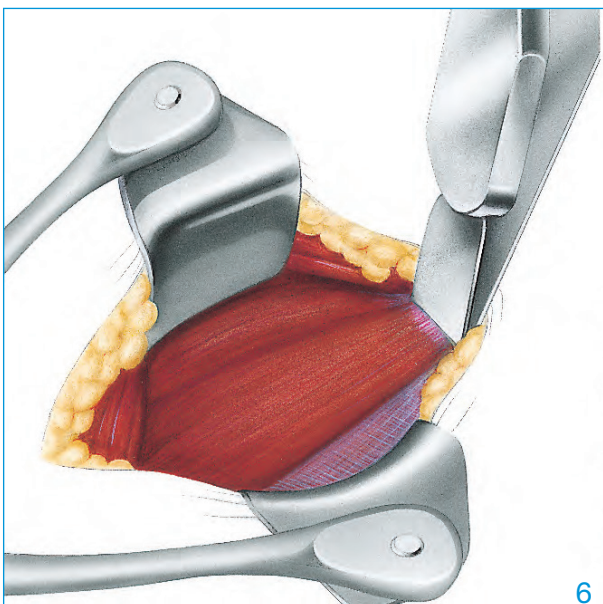At this point the Kölbl Self-Retaining Retractor is used. The longer retractor blade is used to retract the coracobrachial muscle. In muscular patients, transection of the upper border of the Pectoralis major tendon is recommended to extend the access.

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Around 5 mm is chiselled off the tip of the coracoid process, and the latter is rotated medially. The longer Blade of the Retractor can now retract the coracobrachial muscle.

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
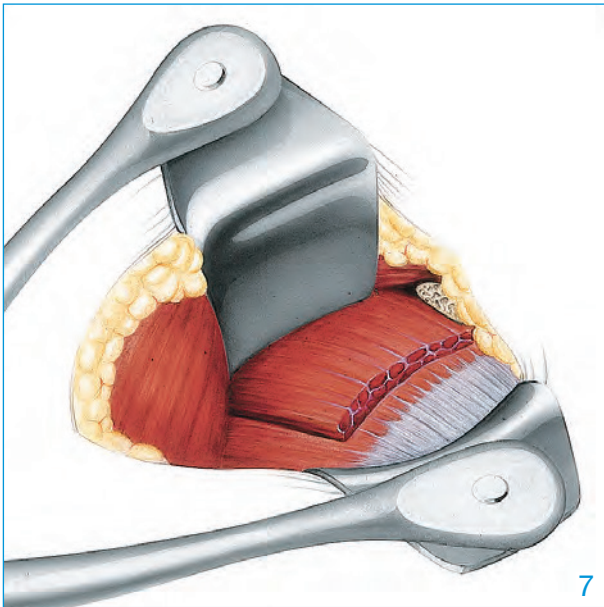
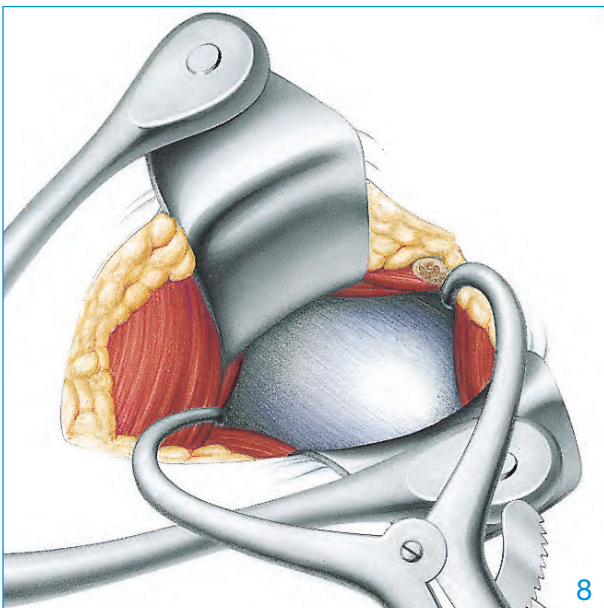
The subscapularis tendon is transected adjacent to the musculotendinous junction from the upper edge onwards. The lower third with the anterior circumflex vessels re-mains in its location. The muscle is longitudinally split in the fibre direction and shifted off the capsule and the neck of the scapula.

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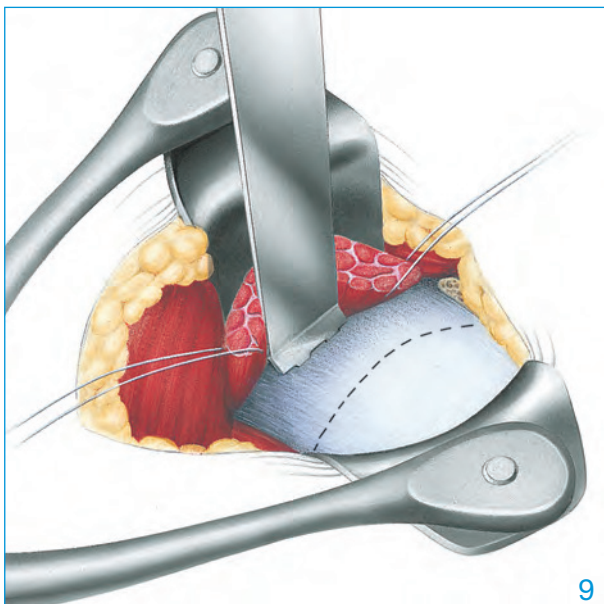Alternatively, the subscapularis muscle and tendon can be longitudinally split in the fibre direction approximately in the middle. The Capsule Retractor is used at this point.

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

The Kölbl Glenoid Retractor at the neck of the scapula retracts the subscapularis. The capsule is incised along the glenoid rim.

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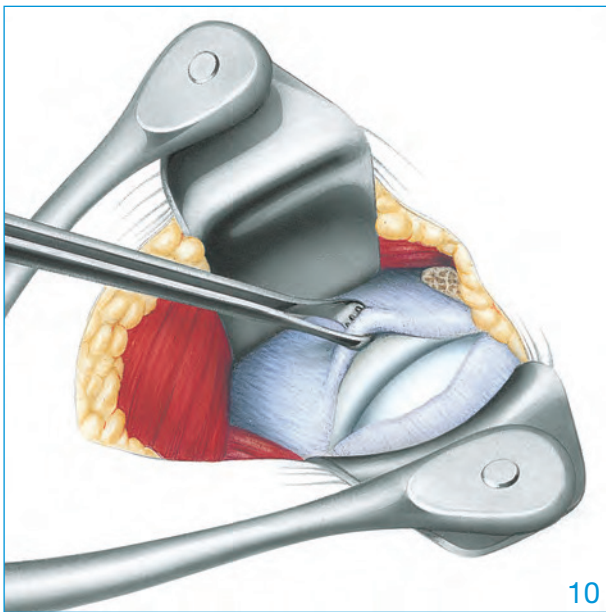
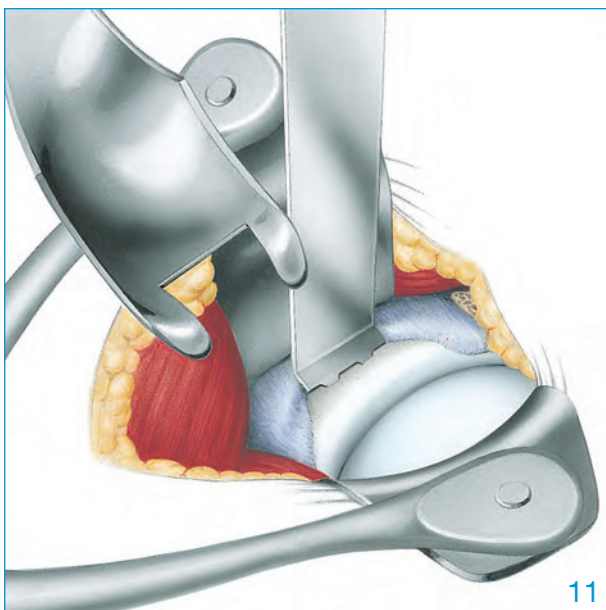
Lifting of the capsule flap reveals the Bankart lesion.

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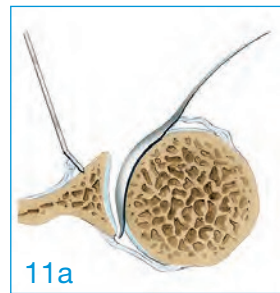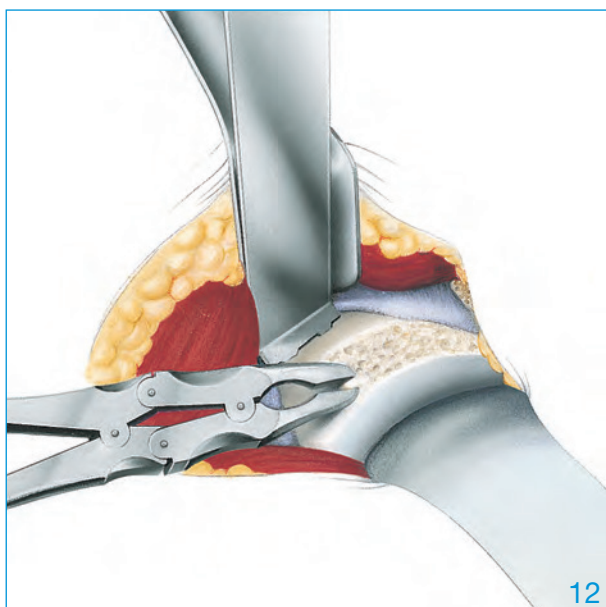The Humeral Head Retractor is introduced into the joint cavity to push aside the humeral head from the glenoid. The medial capsule flap is shifted off the neck of the scapula and re-tracted with the glenoid retractor. The illustration shows the position

of the retractor in the horizontal section. After placement of the humeral head retractor the blade retractor can be removed.

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The anterior rim of the glenoid and the neck of the scapula are completely freed of soft tissue with the slightly curved Rongeur where the capsule is to be reattached. The cortical bone is roughened but should not be removed.

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
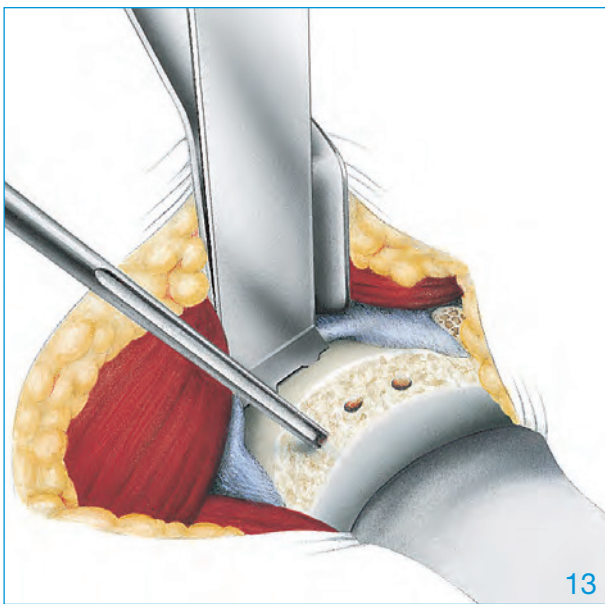
The slim Gouge is used to pierce the cortical bone of the glenoid where the sutures will be placed.

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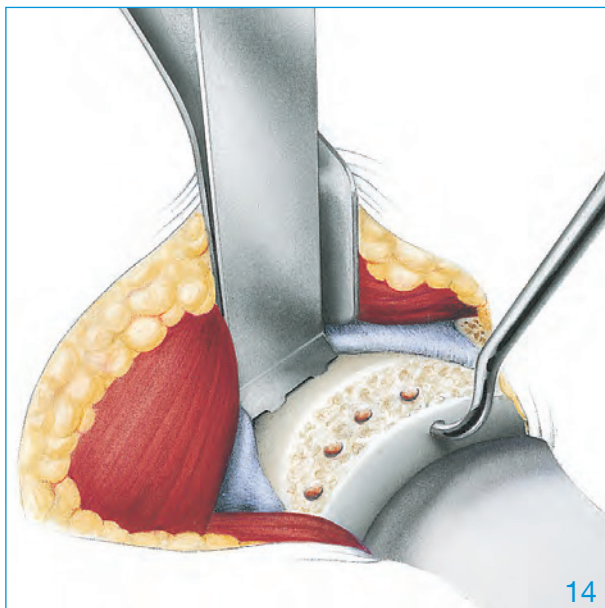
The bone tunnels are made with the curved Awl. The holes are in the articular surface a few millimetres from the rim.

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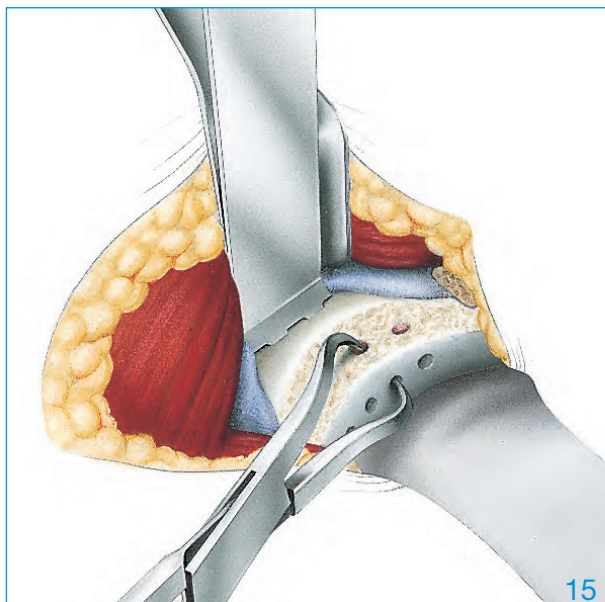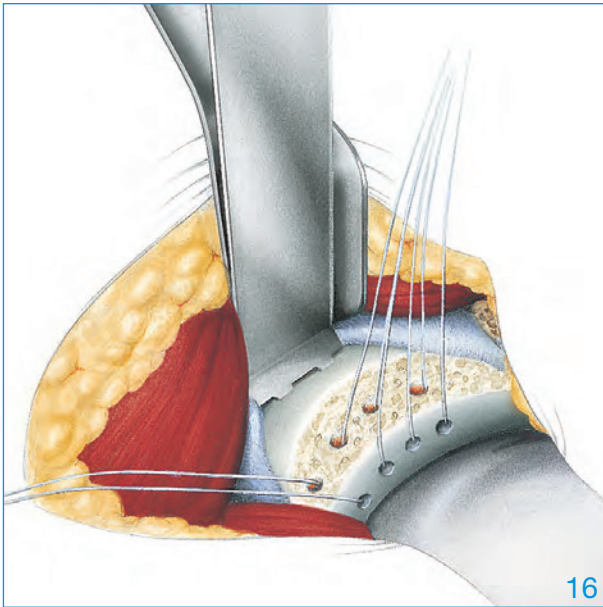
The Perforating Forceps can also be used to make and finish off the bone tunnels. The cross-section clearly shows that the rim of the glenoid must be carefully perforated to prevent the rim breaking off.

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
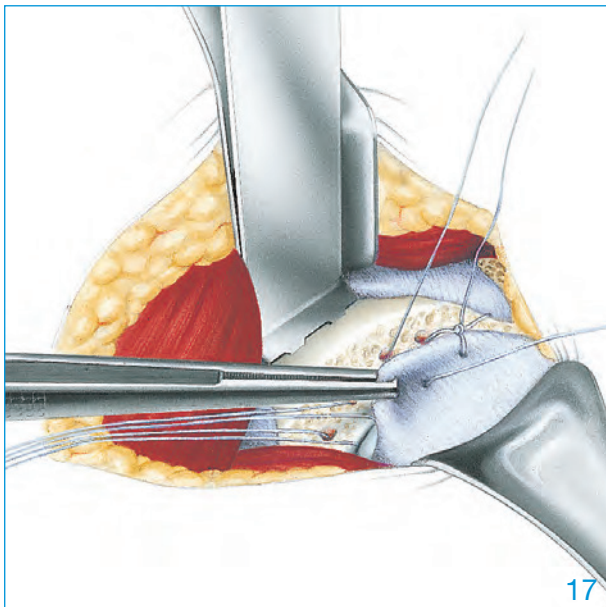
Strong, non-absorbable sutures are passed through the bone tunnels using the special, thick needles. The Humeral Head Retractor is then removed and the blade retractor or the second Richardson Retractor introduced instead.

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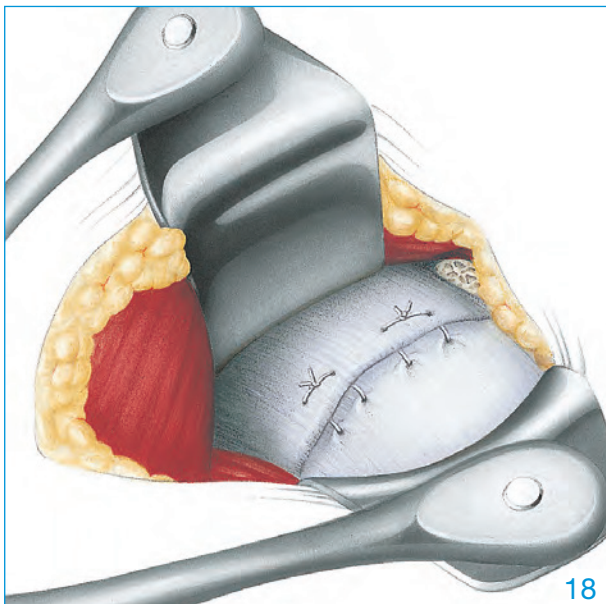
The lateral capsule flap is grasped approximately 4 – 5 mm from the edge and tied securely to the glenoid rim with the strong sutures. The pairs of sutures are not shortened; only one limb of the outer sutures is cut off.

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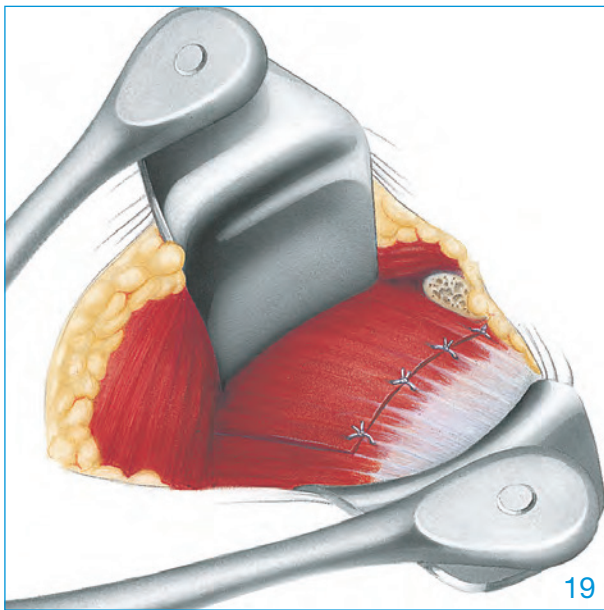
The remaining sutures are used to fix the medial capsule flap in such a way that after tying the sutures it is fixed to the neck of the scapula without any further shortening of the capsule.

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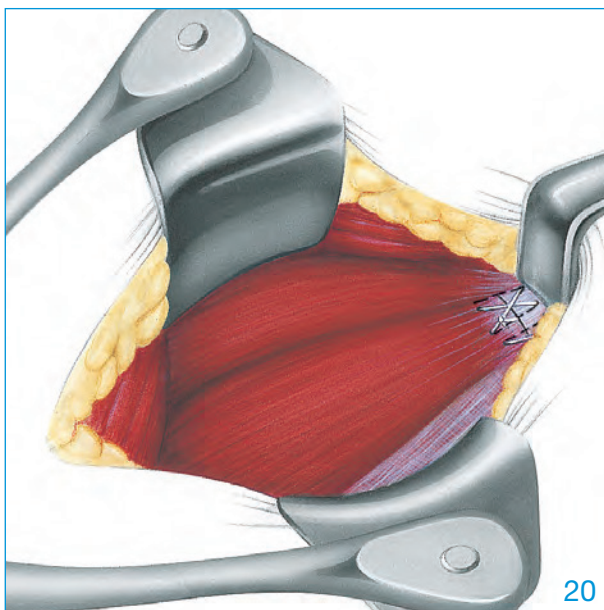Then the subscapularis is fixed carefully.

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The coracoid tip is securely fixed with one or two transosseous far-and-near sutures.

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Please note the following regarding the use of our implants:

- 1. Choosing the right implant is very important.**
The size and shape of the human bone determine the size and shape of the implant and also limit the load capacity. Implants are not designed to withstand unlimited physical stress. Demands should not exceed normal functional loads.
- 2. Correct handling of the implant is very important.**
Under no circumstances should the shape of a finished implant be altered, as this shortens its life span. Our implants must not be combined with implants from other manufacturers.

The instruments indicated in the Surgical Technique must be used to ensure safe implantation of the components.
- 3. Implants must not be reused.**
Implants are supplied sterile and are intended for single use only. Used implants must not be reused.
- 4. After-treatment is also very important.**
The patient must be informed of the limitations of the implant. The load capacity of an implant cannot compare with that of healthy bone!
- 5. Unless otherwise indicated, implants are supplied in sterile packaging.**
Note the following conditions for storage of packaged implants:
 - Avoid extreme or sudden changes in temperature.
 - Sterile implants in their original, intact protective packaging may be stored in permanent buildings up until the "Use by" date indicated on the packaging.
 - They must not be exposed to frost, dampness or direct sunlight, or mechanical damage.
 - Implants may be stored in their original packaging for up to 5 years after the date of manufacture. The "Use by" date is indicated on the product label.
 - Do not use an implant if the packaging is damaged.
- 6. Traceability is important.**
Please use the documentation stickers provided to ensure traceability.
- 7. Further information** on the material composition is available on request from the manufacturer.

Follow the instructions for use!

Waldemar Link GmbH & Co. KG, Hamburg

All content in this catalog, including text, pictures and data, is protected by copyright. Every instance of use, whether in part or in whole and which is not permitted by the Copyright Act, is subject to our prior consent. In particular, this applies to the reproduction, editing, translation, publishing, saving, processing, or passing on of content stored in databases or other electronic media and systems, in any manner or form. The information in the catalogs is solely intended to describe the products and does not constitute a guarantee.

The Surgical Technique described has been written to the best of our knowledge and belief, but it does not relieve the surgeon of his/her responsibility to duly consider the particularities of each individual case.

Unless otherwise indicated, all instruments are made of surgical stainless steel.

Waldemar Link GmbH & Co. KG

Barkhausenweg 10 · 22339 Hamburg
Postfach 63 05 52 · 22315 Hamburg
Telefon: +49 40 53995-0 · Fax: +49 40 5386929
E-Mail: info@linkhh.de · Internet: www.linkhh.de