# Schulterchirurgie Shoulder Surgery

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**CE 0482**

Waldemar Link GmbH & Co. KG

Barkhausenweg 10 · 22339 Hamburg

Tel. +49 40 53995-0 · [info@linkhh.de](mailto:info@linkhh.de)

[www.linkorthopaedics.com](http://www.linkorthopaedics.com)

# LINK<sup>®</sup> Schulterchirurgie

## LINK<sup>®</sup> Shoulder Surgery

|    |                             |                           |
|----|-----------------------------|---------------------------|
| 02 | Beschreibung / Indikationen | Description / Indications |
| 03 | Instrumentarium             | Instrument Set            |
| 05 | Instrumente                 | Instruments               |
| 14 | Zusätzliche Instrumente     | Additional Instruments    |
| 17 | Zubehör                     | Accessories               |
| 18 | OP-Technik                  | Surgical Technique        |
|    | Wichtige Hinweise           | Important Information     |

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The goal of operative treatment of posttraumatic recurrent glenohumeral luxation is to restore stability by repairing the traumatic lesion, returning structures to their anatomic locations and restoring the physiological range of motion.

The surgical procedure by which this can be achieved is the

- **Bankart Procedure**

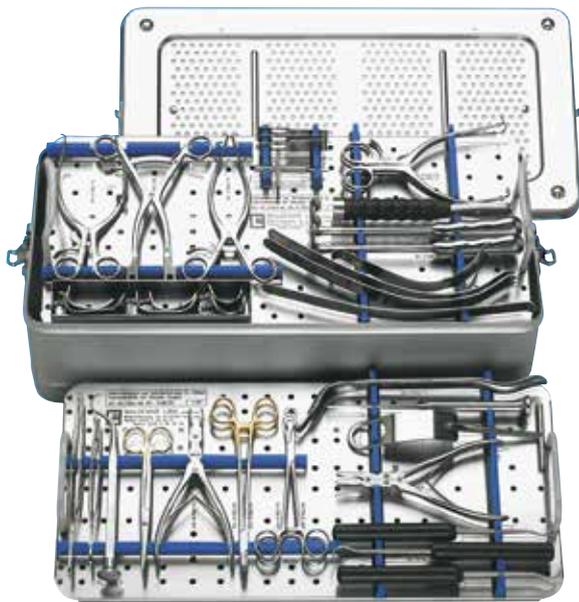
In this procedure, it is necessary to expose the anatomic structures individually and to suture them in the original location after reattachment of the capsule to the anterior rim of the glenoid cavity. This is the only way to obtain equilibrium of the muscle actions with full joint mobility.

A specially designed instrument set is required to perform a quick and tissue-sparing procedure. The instrument set featured in this Surgical Technique meets these requirements and can also be used for many other forms of shoulder surgery.

Other **indications** for the Shoulder Surgery instrument set:

- Capsuloplasties for multidirectional instabilities
- Subacromial decompression (Neer procedure)
- Calcium excision in the case of Calcific Tendinitis
- Shoulder replacement, fracture repair

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Percussion Awls  
with square point and striking head,  
curved, 220 mm

| Art.-Nr.<br>Item No. | Größe     | Size    |
|----------------------|-----------|---------|
| <b>64-4160/20</b>    | klein     | small   |
| <b>64-4160/05*</b>   | mittel    | medium  |
| <b>64-4160/22</b>    | groß      | large   |
| <b>64-4160/23</b>    | extragroß | x-large |

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\* Size "medium" included in Instrument Set

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For transosseous humeral sutures. Use in conjunction with trocar point needles page 20.

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Penetrating Awl with square point,  
curved, 235 mm

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Crochet Hook Suture Passer,  
modified version:

- shaft area 20 mm reduced
- handle 30 mm reduced
- total length: 185 mm

**64-4160/08**

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Hollow Lumen Suture Passer,  
curved, 235 mm

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Hollow Lumen Suture Passer,  
modified version:

- shaft area 20 mm reduced
- handle 30 mm reduced
- total length: 185 mm

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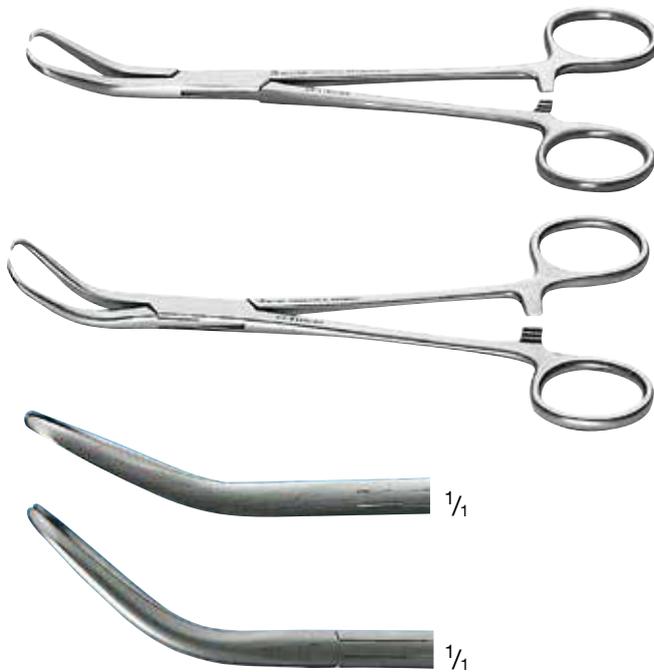
Penetrating Gouge, 3 mm wide, 220 mm.  
For perforating the glenoid rim or neck of  
the scapula.

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Humerus head retractor, forked, inward-  
angled blunt tips, strongly curved, 180 mm

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Fukuda-Kujat's Humeral Head  
Retractor, 35 mm, wide blade, 225 mm

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Richardson Retractor,  
20 mm wide x 55 mm deep, 235 mm

For retracting the tissue when opening  
the deltopectoral interval.

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Kölbel Glenoid Retractor,  
double-pronged, angled tips, 300 mm

| Art.-Nr.<br>Item No. | Breite<br>Width | Version                          |
|----------------------|-----------------|----------------------------------|
| <b>64-4160/13</b>    | 18 mm           | gerade<br>straight               |
| <b>64-4160/32</b>    | 15 mm           | stark gebogen<br>strongly curved |
| <b>64-4160/33</b>    | 23 mm           | stark gebogen<br>strongly curved |

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For opening the subacromial space by depressing the humeral head.

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For use as a soft tissue lever to approach the coracoid.

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For exposing the glenoid in Bankart operations and implantations.

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Kölbl Soft Tissue Retractor, single-pronged, straight, very fine tips, 190 mm

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Kölbl Soft Tissue Retractor, single-pronged, angled, 190 mm

For retraction when using the deltopectoral approach.

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Kölbl Soft Tissue Retractor, double-pronged, angled, 190 mm

For retraction when using the deltopectoral approach.

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Osteotome Set  
with three exchangeable blades  
(10, 16 and 25 mm) and one key,  
200 mm

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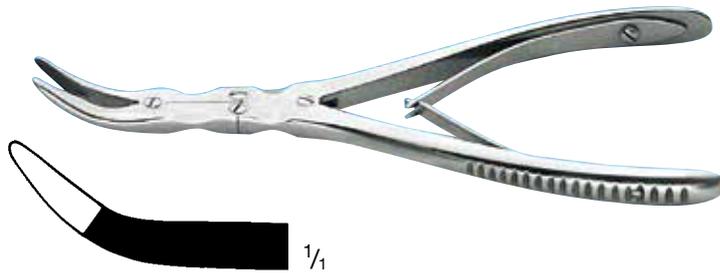
Bone Nibbling Rongeur,  
double-action, slightly curved, 5 mm  
jaw width, 190 mm

For roughening the rim of the glenoid  
at the side of the Bankart lesion and  
for preparing the humerus for rotator  
reconstruction. Also for modeling the  
acromion.

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Köbel Tenectomy Rongeur, double-action, strongly curved, 2.5 x 5.0 mm jaw width, 180 mm

For removing tendon residue and for deepening the neck of the humerus in tendon refixation.

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Skid Lever, (bone lever and skid curette), double-ended, 280 mm

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Köbel Self-Retaining Glenoid Retractor, with ball snap-fit, slightly curved, with 3 pairs of blades, 200 mm

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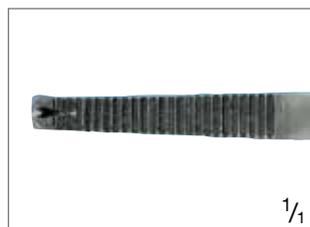
Capsule and Tendon Scissors  
curved, with serrated carbide inserts,  
170 mm

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Kölbel Tissue Forceps  
1 x 2 teeth and serrated jaws, heavy  
pattern can securely grasp capsular  
membrane, 210 mm

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Bonney Thumb Forceps,  
1 x 2 teeth and serrations,  
175 mm

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Heaney Needle Holder  
curved, carbide inserts, 205 mm

For transosseous suturing of the capsule and for passing the strong Mayo needle through the bone canals of the glenoid rim

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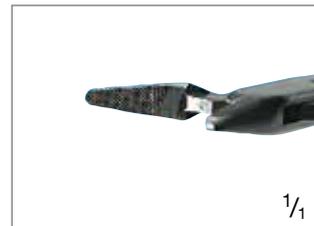
New Orleans Needle Holder  
carbide inserts, 180 mm

For tendon and transosseous suturing during reconstruction of the rotator cuff.

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Kölbel Subscapular Spreader,  
double-pronged, angled arms, 190 mm

For retraction when using the delto-pectoral approach.

| Art.-Nr.<br>Item No. | Maße<br>Dimensions | Ausführung<br>Version |
|----------------------|--------------------|-----------------------|
| <b>64-4160/18</b>    | 20 x 12 mm         | klein<br>small        |
| <b>64-4160/19</b>    | 25 x 15 mm         | groß<br>large         |

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Dahmen-Lexer Gouge  
25 mm wide, 220 mm

For partial resection of the acromio-  
clavicular joint.

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Mallet  
with linen phenolic handle,  
Ø 30 mm, 480g, 240 mm

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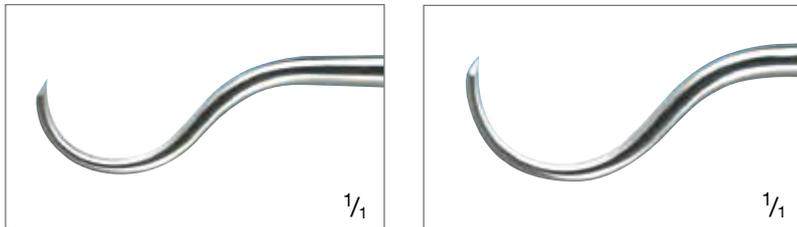
Humerus Head Retractors,  
american model

| Art.-Nr.<br>Item No. | Breite (Zoll)<br>Width (inch) | Länge<br>Length |
|----------------------|-------------------------------|-----------------|
| 64-4163/01           | 3 / 8"                        | 255             |
| 64-4163/02           | 1 / 2"                        | 255             |
| 64-4163/03           | 3 / 4"                        | 255             |
| 64-4163/04           | 1"                            | 355             |

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Penetrating Awls,  
american model, curved, 235 mm


| Art.-Nr.<br>Item No. | Radius  |
|----------------------|---------|
| 64-4161/61           | 9,5 mm  |
| 64-4161/62           | 11,5 mm |

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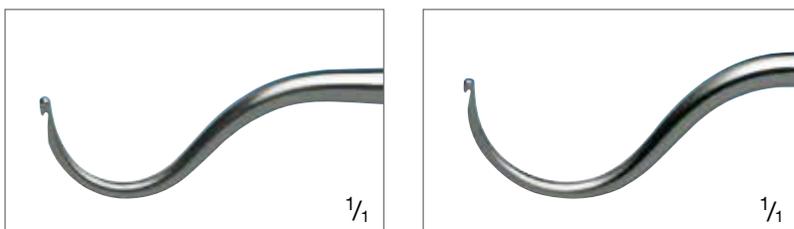Crochet Hook Suture Passer,  
american model, curved, 235 mm

| Art.-Nr.<br>Item No. | Radius  |
|----------------------|---------|
| 64-4161/71           | 9,5 mm  |
| 64-4161/72           | 11,5 mm |

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The awls and suture passers 64-4161/61, 62 and 64-4161/71, /72 differ from the instruments 64-4160/06 and /07 in the size of their tips. The curvature of these instruments corresponds to the curvature of the percussion awls 64-4160/20 and /22.

Nicht im Instrumentarium 64-4158/01 enthalten - not included in Instrument Set 64-4158/01

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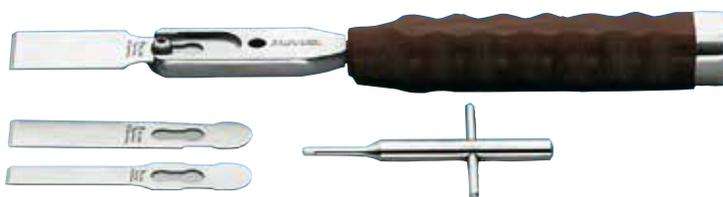
Köbel Self Retaining Glenoid Retractor, with ball snap-fit, slightly curved, with 3 pairs of blades, 200 mm

Bestehend aus:  
Consisting of:

| Art.-Nr.<br>Item No. | Ausführung<br>Version                            | Tiefe x Breite<br>Depth x Width |
|----------------------|--|---------------------------------|
| <b>64-4161/02</b>    | Spreizer ohne Valven<br>Retractor without blades | 200 mm                          |
| <b>64-4161/07</b>    | Valven kurz<br>Blades, short                     | 36 x 18 mm                      |
| <b>64-4161/08</b>    | Valven lang<br>Blades, long                      | 53 x 18 mm                      |
| <b>64-4161/09</b>    | Valven extralang<br>Blades, extra long           | 68 x 18 mm                      |

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
Individual components for Osteotome Set  
**(64-4159/16)**


- 64-4159/93** Griff, einzeln inkl. Schraube  
Handle, single including Screw
- 64-4159/94** Auswechselbare Klingen,  
3 Stück im Satz (10, 16, 25 mm)  
Replacement Blades,  
Set of 3 (10, 16, 25 mm)
- 64-4159/95** Auswechselbare Klinge, 10 mm  
Replacement Blade, 10 mm
- 64-4159/96** Auswechselbare Klinge, 16 mm  
Replacement Blade, 16 mm
- 64-4159/97** Auswechselbare Klinge, 25 mm  
Replacement Blade, 25 mm
- 64-4159/98** Ersatz Schraube  
replacement of Screw
- 64-4159/99** Schlüssel  
Key

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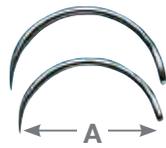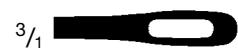The Needles are ideal for transosseous sutures, i.e. for passing No. 2 and No. 3 sutures through the rim of the glenoid.

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

Trocar Point Needles  
with standard eye, pack of 24

| Trokarnadel<br>Trocar Point Needle |                   | Ganzmetallpfriem<br>Percussion Awl |   |
|------------------------------------|-------------------|------------------------------------|---|
|                                    | <b>64-4160/50</b> | entspricht<br>corresponds to       | <b>64-4160/20</b> klein/small<br>       |
|                                    | <b>64-4160/51</b> | entspricht<br>corresponds to       | <b>64-4160/05</b> mittel/medium<br>     |
|                                    | <b>64-4160/52</b> | entspricht<br>corresponds to       | <b>64-4160/22</b> groß/large<br>        |
|                                    | <b>64-4160/53</b> | entspricht<br>corresponds to       | <b>64-4160/23</b> extragroß/x-large<br> |

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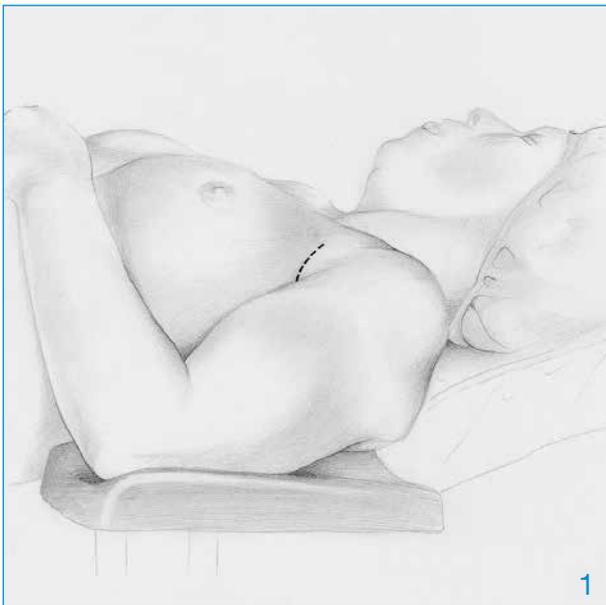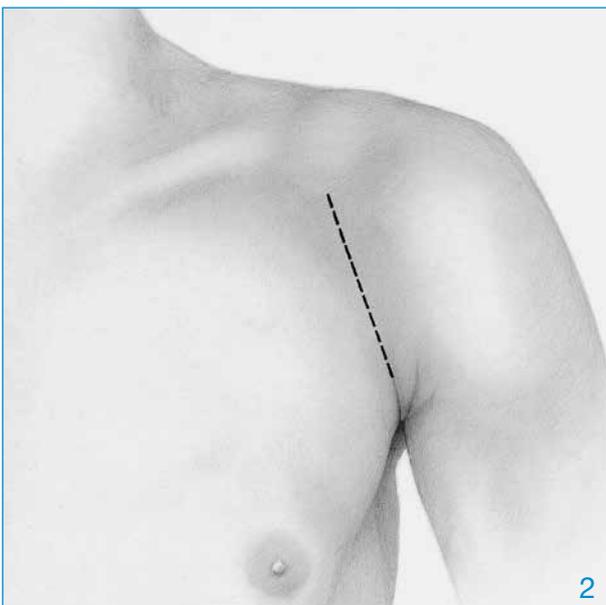
The upper arm is positioned on an arm rest that extends no further than the elbow. If the table is wide, the upper arm is positioned similarly on a folded drape leaving the shoulder exposed. This allows the surgeon to hold the arm if so desired and the humeral head to be shifted to dorsal. No cushion is placed under the scapula; by placing a drape, folded repeatedly to a thickness of 1 – 1.5 cm, medial of the medial border, the scapula can be pushed outwards, which can sometimes be advantageous.

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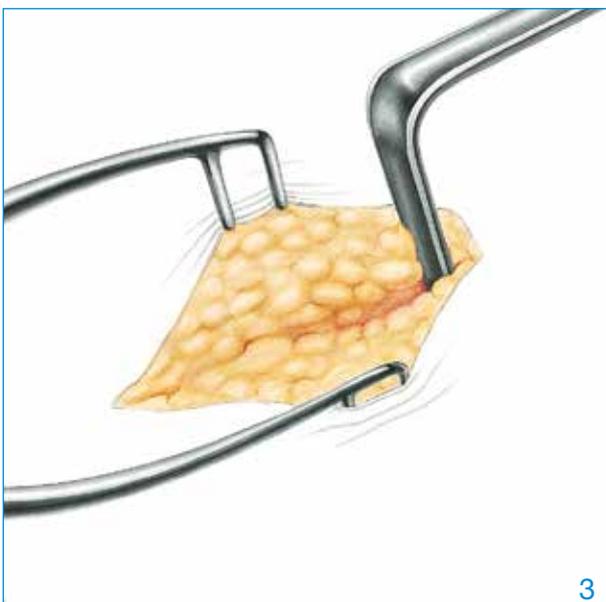
The skin incision is made from the coracoid to one of folds above the anterior edge of the axilla. The skin incision does not have to be as long for most women.

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
Retraction begins with the double-pronged Kölbl Soft Tissue Retractor and a Richardson Retractor.

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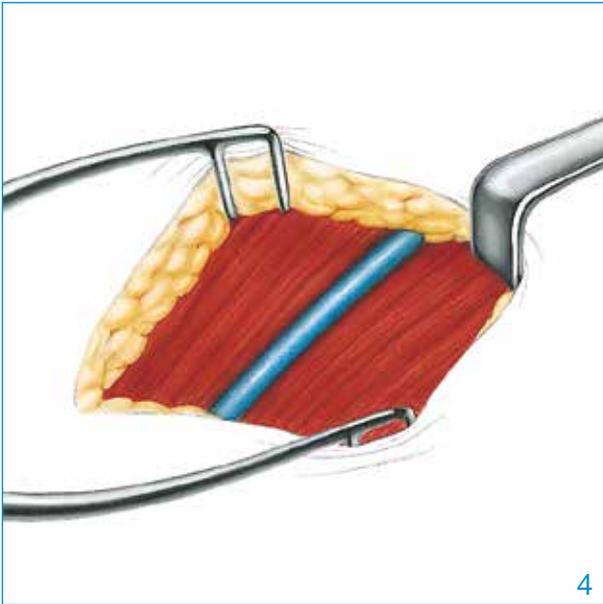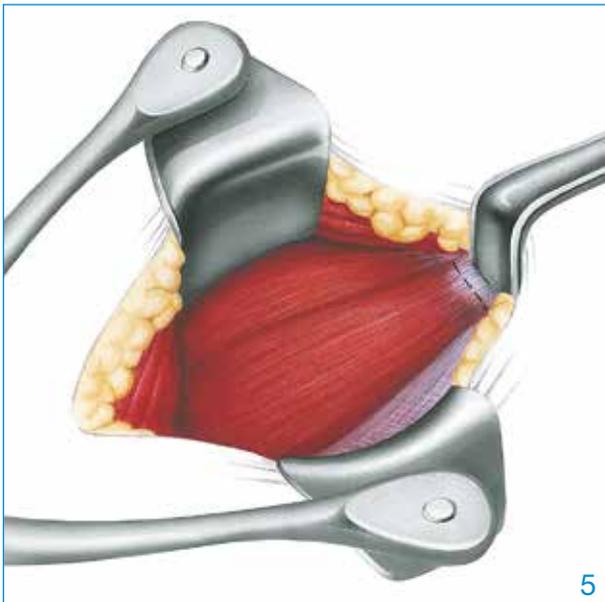
The cephalic vein runs obliquely to the skin incision from top right to bottom left. Access is via the deltoideopectoral sulcus with the cephalic vein to lateral. This exposes the coracoid and the lateral border of the coracobrachial muscle.

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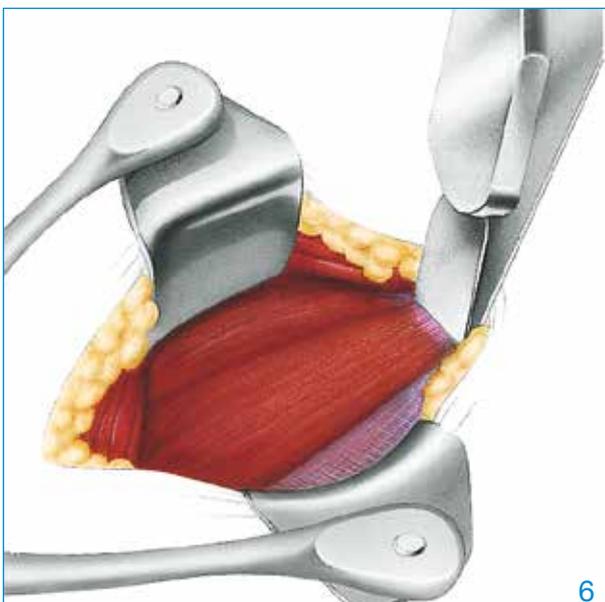At this point the Kölbl Self-Retaining Retractor is used. The longer retractor blade is used to retract the coracobrachial muscle. In muscular patients, transection of the upper border of the Pectoralis major tendon is recommended to extend the access.

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Around 5 mm is chiselled off the tip of the coracoid process, and the latter is rotated medially. The longer Blade of the Retractor can now retract the coracobrachial muscle.

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
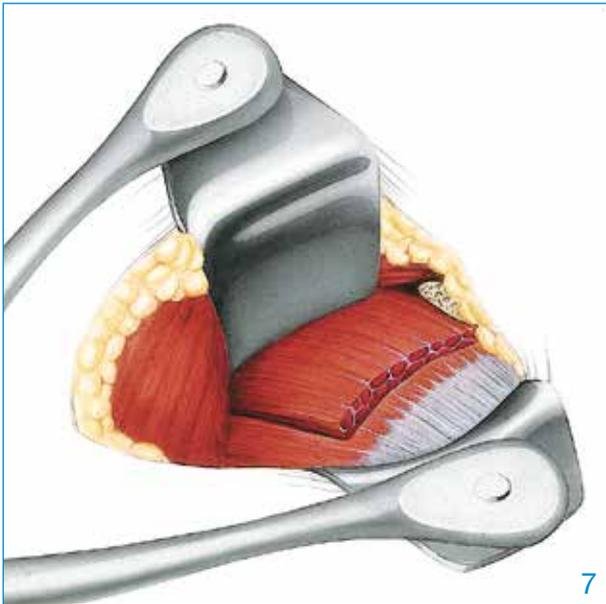
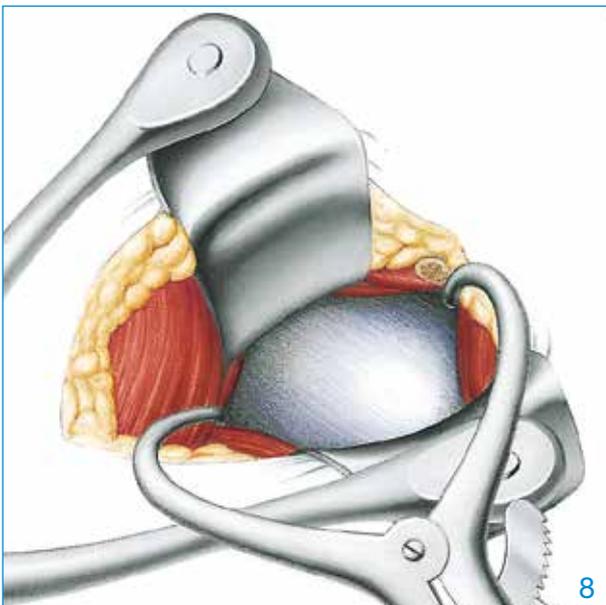
The subscapularis tendon is transected adjacent to the musculotendinous junction from the upper edge onwards. The lower third with the anterior circumflex vessels re-mains in its location. The muscle is longitudinally split in the fibre direction and shifted off the capsule and the neck of the scapula.

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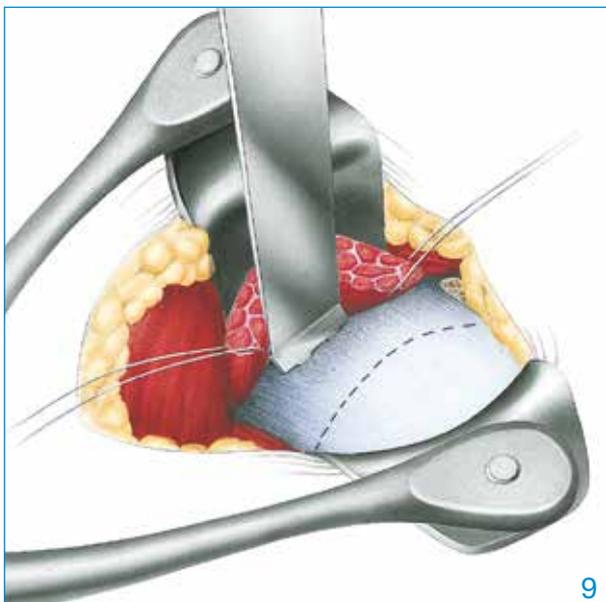Alternatively, the subscapularis muscle and tendon can be longitudinally split in the fibre direction approximately in the middle. The Capsule Retractor is used at this point.

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

The Kölbl Glenoid Retractor at the neck of the scapula retracts the subscapularis. The capsule is incised along the glenoid rim.

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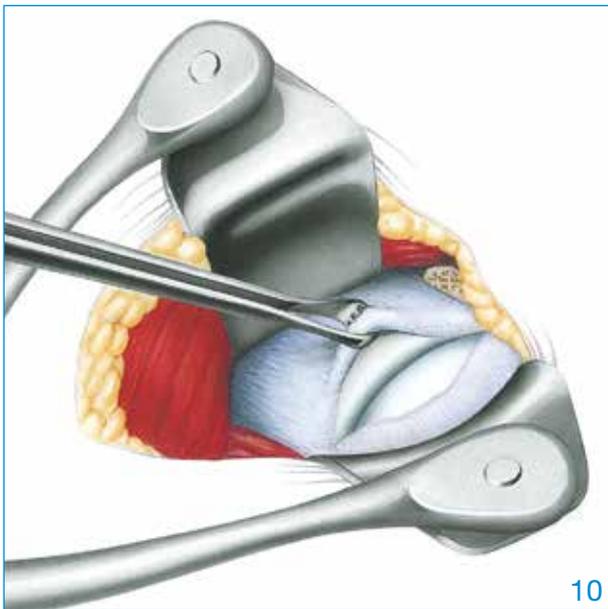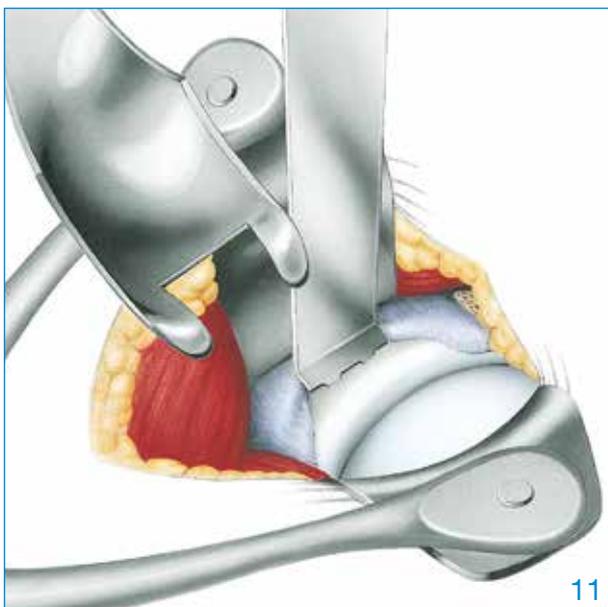
Lifting of the capsule flap reveals the Bankart lesion.

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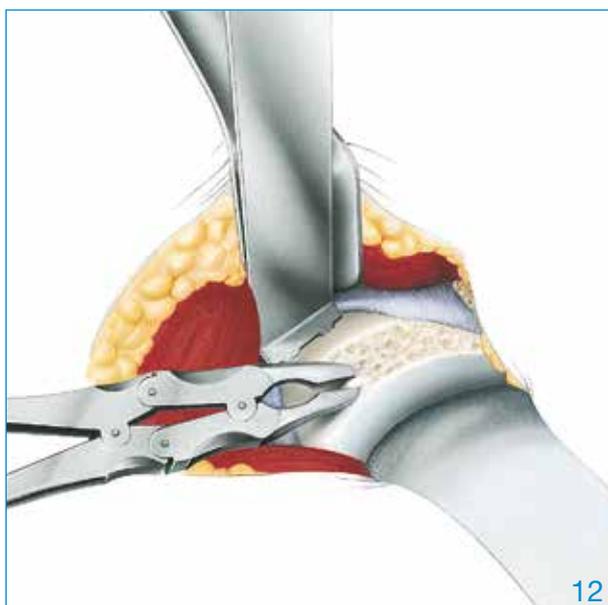The Humeral Head Retractor is introduced into the joint cavity to push aside the humeral head from the glenoid. The medial capsule flap is shifted off the neck of the scapula and re-traced with the glenoid retractor. The illustration shows the position

of the retractor in the horizontal section. After placement of the humeral head retractor the blade retractor can be removed.

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The anterior rim of the glenoid and the neck of the scapula are completely freed of soft tissue with the slightly curved Rongeur where the capsule is to be reattached. The cortical bone is roughened but should not be removed.

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
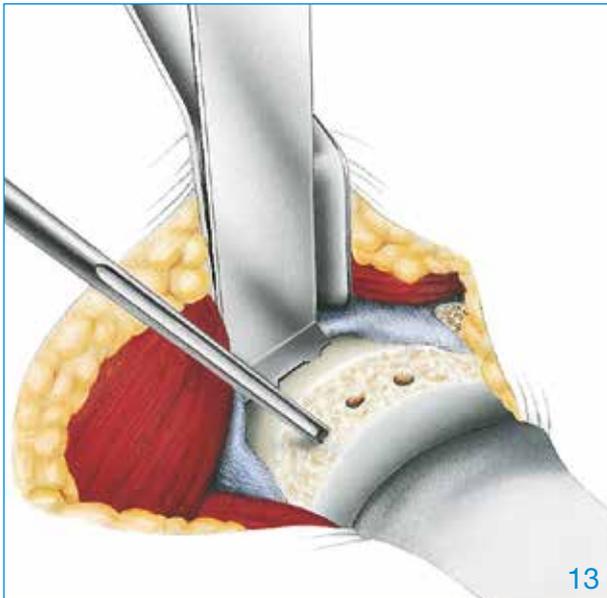
The slim Gouge is used to pierce the cortical bone of the glenoid where the sutures will be placed.

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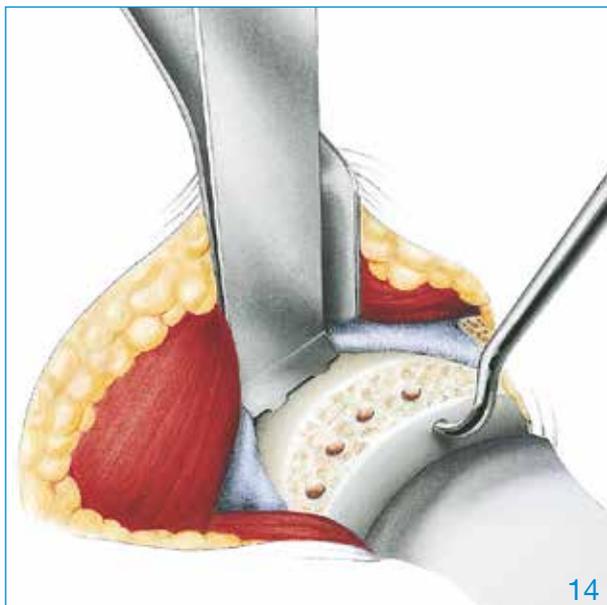
The bone tunnels are made with the curved Awl. The holes are in the articular surface a few millimetres from the rim.

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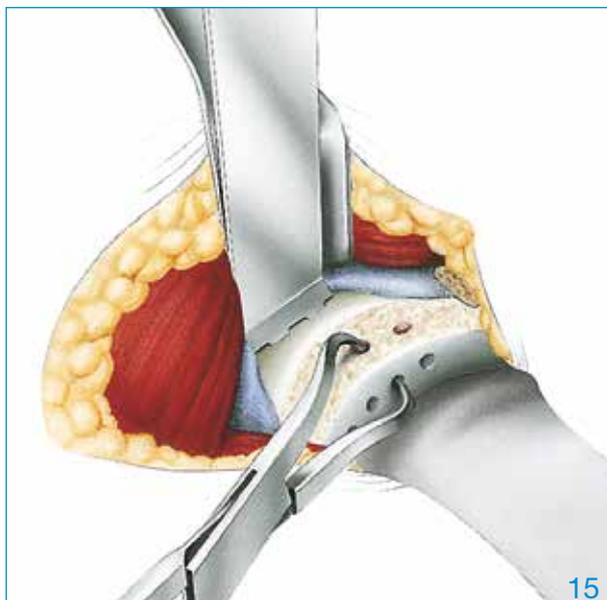
The Perforating Forceps can also be used to make and finish off the bone tunnels. The cross-section clearly shows that the rim of the glenoid must be carefully perforated to prevent the rim breaking off.

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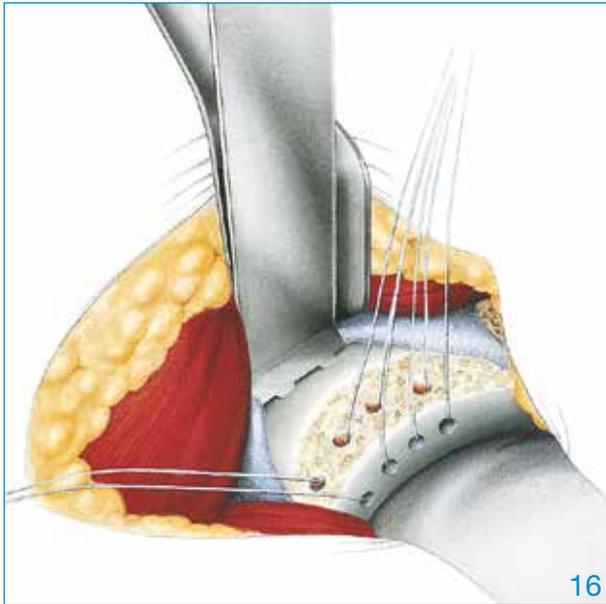
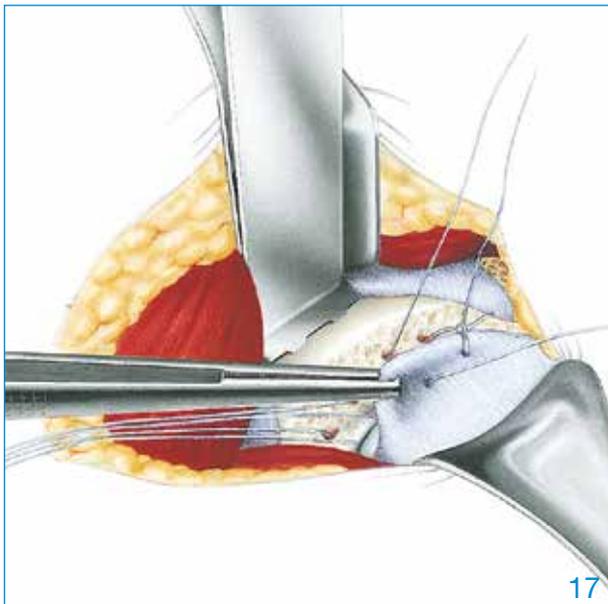
Strong, non-absorbable sutures are passed through the bone tunnels using the special, thick needles. The Humeral Head Retractor is then removed and the blade retractor or the second Richardson Retractor introduced instead.

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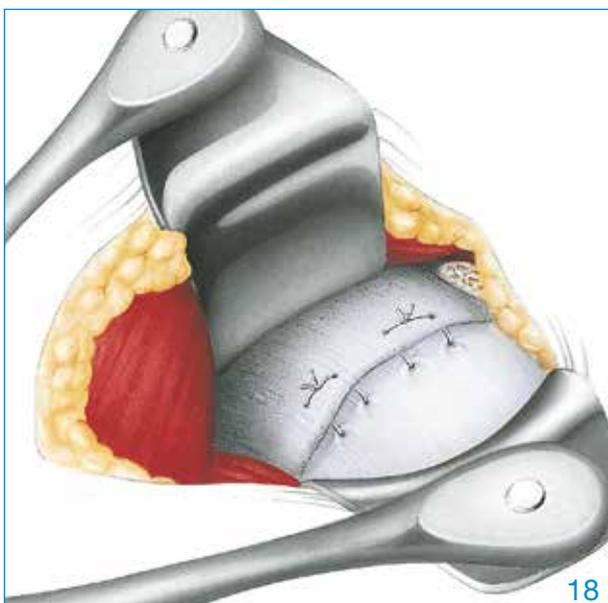
The lateral capsule flap is grasped approximately 4 – 5 mm from the edge and tied securely to the glenoid rim with the strong sutures. The pairs of sutures are not shortened; only one limb of the outer sutures is cut off.

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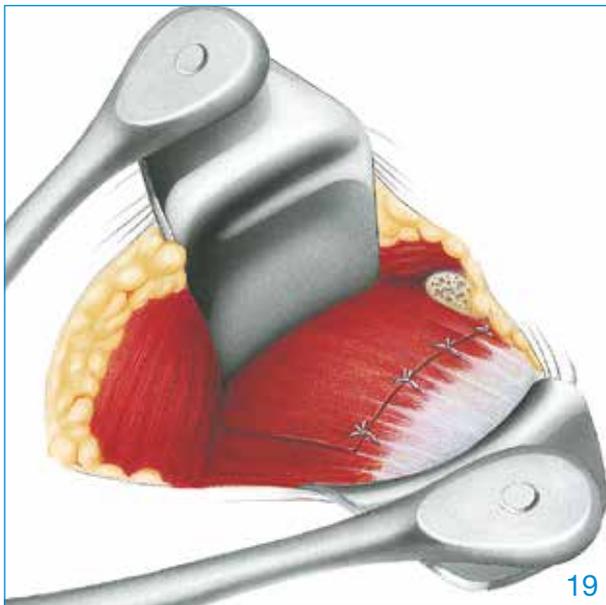
The remaining sutures are used to fix the medial capsule flap in such a way that after tying the sutures it is fixed to the neck of the scapula without any further shortening of the capsule.

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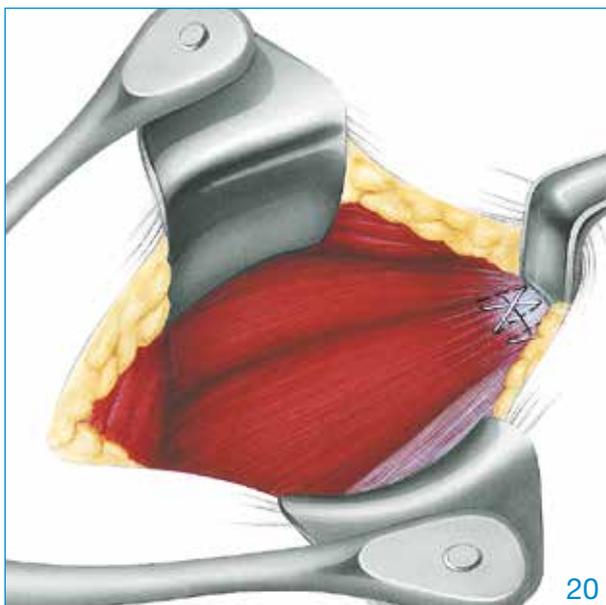Then the subscapularis is fixed carefully.

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The coracoid tip is securely fixed with one or two transosseous far-and-near sutures.

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Waldemar Link GmbH & Co. KG, Hamburg